


ComfortBand PLUS™ Wristband Instructions

1



Lay sheet on a flat surface. Hold top left corner of sheet and gently push out wristband to separate.

2

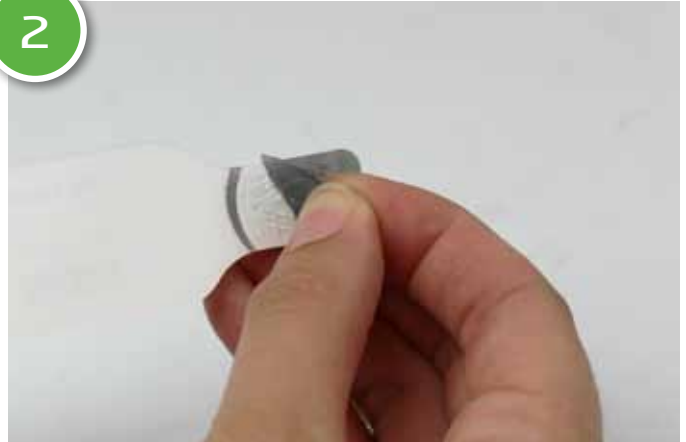

Remove gray tab on back of wristband to expose adhesive.

3



Wrap wristband around patient's wrist and secure by pressing the adhesive portion to strap.

OPTIONAL STEP



Optional: STAT label may be used on the wristband (over the adhesive closure) or on documentation, according to your hospital protocol.

Communicate multiple color code alerts on ComfortBand PLUS™ easily and effectively with Ident-Alert® Clasps:

- Meet standardization requirements by combining both colors and words
- Easy to read alerts
- Variety of stock colors and imprints
- Latex and Phthalate-Free



Ident-Alert® 8621
solid, colored, plastic snaps with printed alert text