



Ident-Alert® IV Port Clips - Instructions for use

These color coded clips are designed specifically for IV injection ports to act as a last step visual cue and reminder for medication warnings or allergies. They help prevent potentially lethal medication administration errors by alerting the anesthesiologist right at the site of the drug injection.

The Ident-Alert IV Port Clips come in 6 different colors.

Part Number	Description
IVPORT-CLIP-R	IDENT-ALERT™ IV PORT ALERT CLIP, RED
IVPORT-CLIP-P	IDENT-ALERT™ IV PORT ALERT CLIP, PUR
IVPORT-CLIP-B	IDENT-ALERT™ IV PORT ALERT CLIP, BLU
IVPORT-CLIP-O	IDENT-ALERT™ IV PORT ALERT CLIP, ORG
IVPORT-CLIP-G	IDENT-ALERT™ IV PORT ALERT CLIP, GRN
IVPORT-CLIP-W	IDENT-ALERT™ IV PORT ALERT CLIP, WHITE

1. Identify port(s) that need identification
2. Obtain the right colored clip for use. They can be written on with a permanent marker or a ball point pen.
3. Clip location differs by brand. Place clip securely onto the correct location – see diagram below.



4. Once the IV bag is done, always throw away the clip with the line and the bag. **The product is not meant to be reused due to contamination concerns.**

Learn more pdchealthcare.com/IVportclips.