


How to Use Kleen-Print® Footprinter

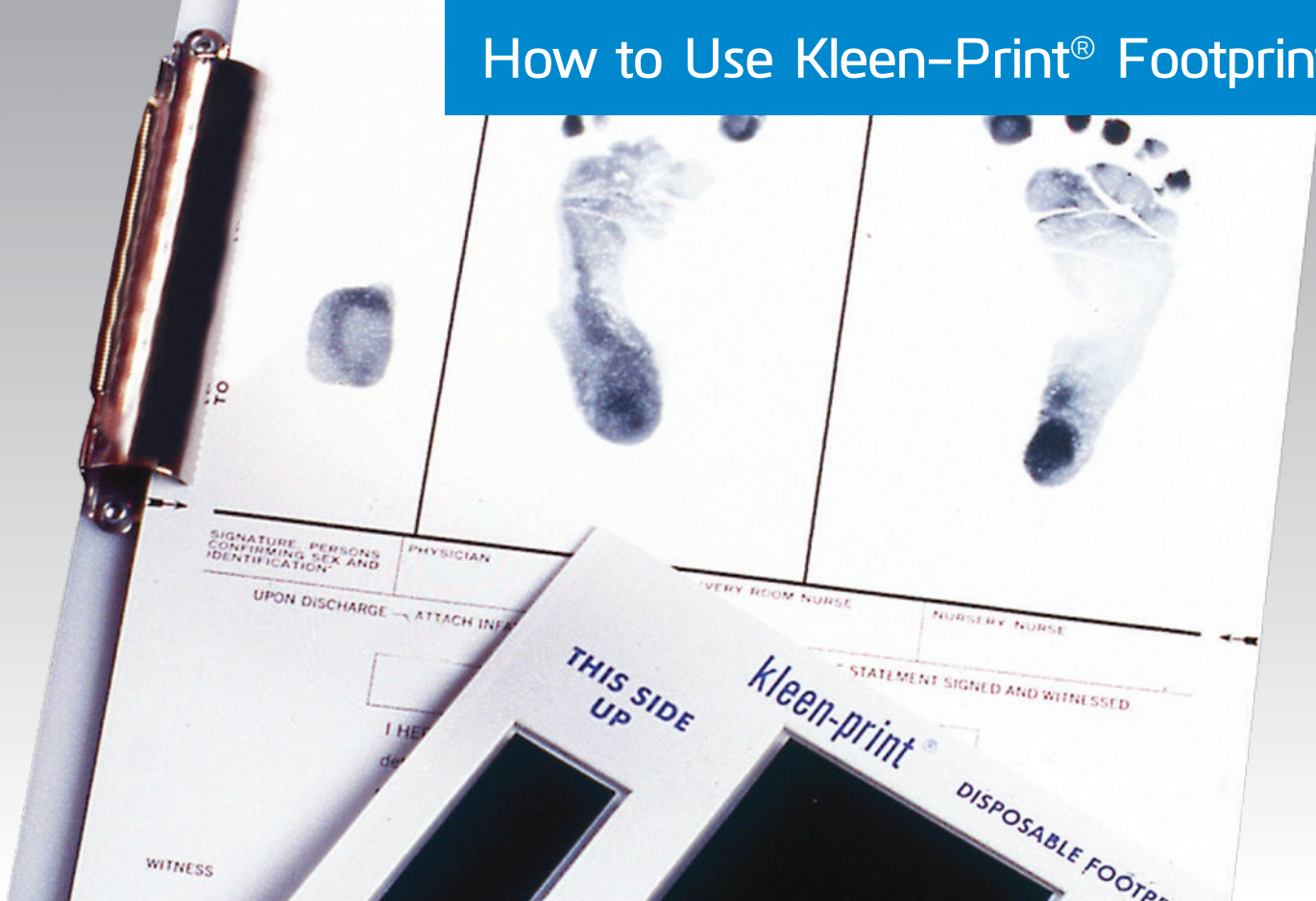

1. Place Kleen-Print® frame printed side up on top of the identification form. Clean and dry infant's foot before taking print.



2. To make print "walk" infant's left foot (from heel to toe) on the printing form using very light pressure. Repeat process with right foot. Do not move infant's foot while pressing down.



3. Make print of mother's right index finger, rolling from side to side while pressing lightly. Discard frame and dust prints with baby powder to prevent smearing.

27770 N. Entertainment Dr., Ste. 200
Valencia, CA 91355
Tel 800.435.4242 or 661.257.0233
Fax 800.321.4409
www.pdchealthcare.com
E-mail info@pdchealthcare.com

Where positive identification
meets safe, efficient patient care.



A Brady Business  BRADY

HC-530-1215