



ScanBand® Instructions for Use

- 1 Print information and tear wristband from the roll.
 - 1A Printable surface area is 3.75" starting at black sensor bar on the reverse side of the band and extending toward the adhesive closure.
- 2 With information facing down, remove tab from the end of the wristband.

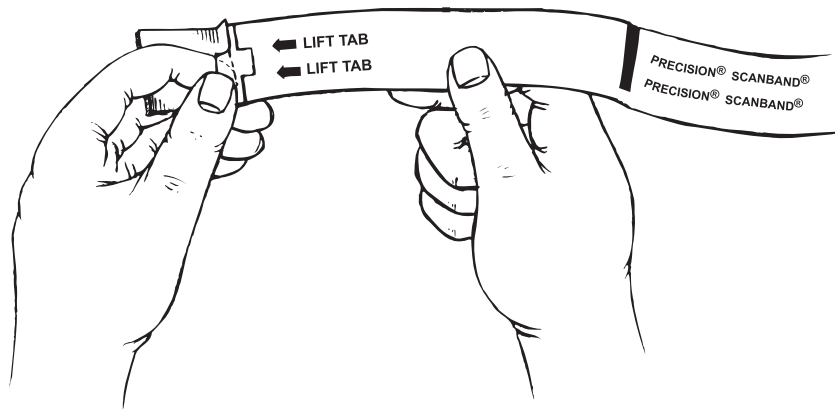

- 3 Adjust the wristband around patient's wrist, press down on adhesive closure to fasten and secure.

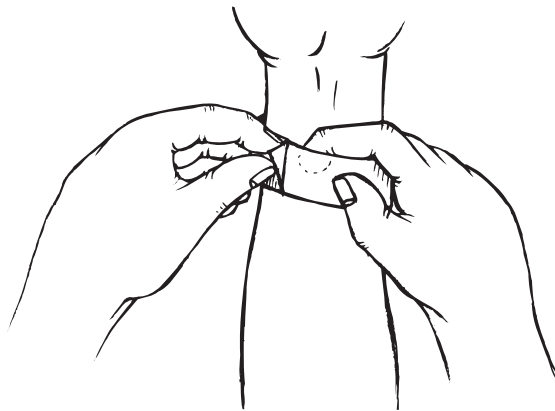

- 4 PDC recommends cleaning printer rollers regularly. Check with your printer manufacturer for service procedure.



13880 Del Sur Street | San Fernando, CA 91340-3490 U.S.A.

Tel: 800.772.1122 • 818.897.1111 | Fax: 818.686.9314 | E-mail: info@pdcorp.com

W-7045-45-13 N/C

www.pdc-healthcare.com