




- 1** Fill out information on insert card, specimen tube label and corresponding labels.

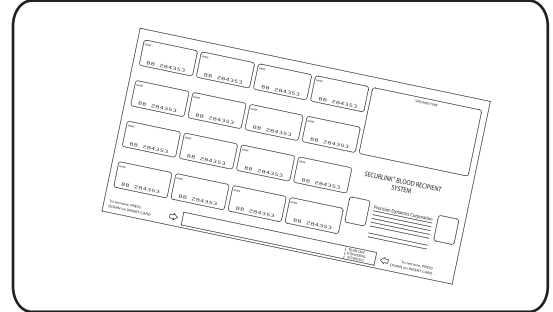


- 2** Remove insert card from the bottom of label sheet and insert into wristband.

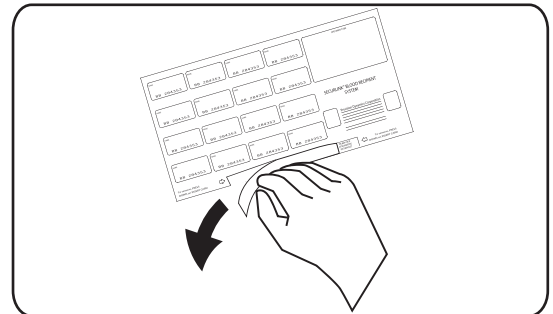

- 3** Size band around patient's wrist and close plastic snap, cut off any excess band.

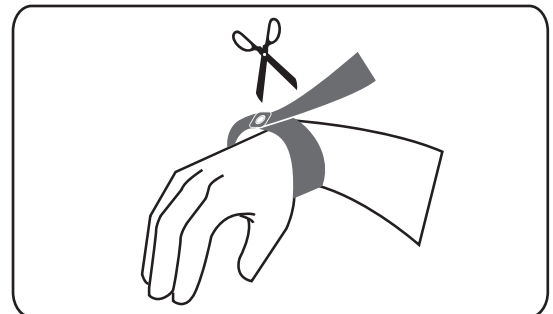

- 4** Remove specimen tube label from sheet. Then, attach label to draw tube.

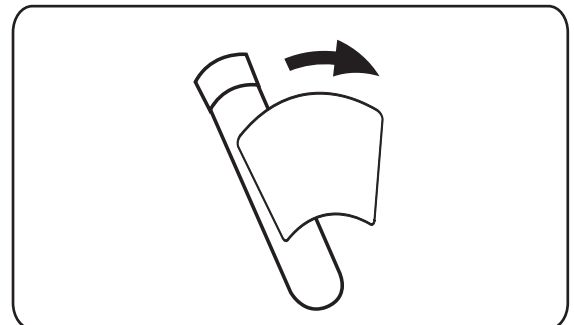

- 5** Use the remaining labels for crossmatching and charting purposes.

Precision Dynamics Corporation
13880 Del Sur Street
San Fernando, California 91340-3490 U.S.A.
Tel: 818.897.1111 • Fax: 818.686.9314
Email: info@pdcorp.com • www.pdc-healthcare.com

For more information call Customer Care at
800-772-1122
6:30am to 4:30pm Pacific Time