

## Check out other Blood Safety Labeling and Patient ID Solutions from PDC


### Reduce Errors and Improve Patient Safety with ISBT Compliant Labels

PDC can help you meet the ISBT Blood Bank standards with our complete line of blank and pre-printed labels.



### Get High Impact Communication with Highly Visible MedVision® Labels


Choose from a variety of preprinted fluorescent labels to communicate important alerts. Customize your own label to meet your specific needs.

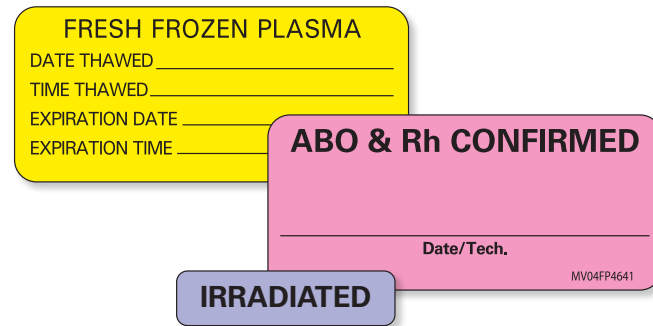

### Streamline Lab Workflow with LIS Labels Designed for Leading Laboratory Information Systems

No matter the LIS and printers you use, we have the largest selection of compatible labels. Enhancing optimal workflow efficiencies, our robust LIS label line includes a variety of solutions for Sunquest, Cerner, MEDITECH, SoftLab, and others. Plus, our V-Notch Secondary Labels feature a patented design for use with BD Vacutainer® Plus Plastic Tubes for correct and aligned label placement every time.

As experts in labels and media, we'll help you choose the right products for your unique applications. Call us at 800.435.4242.

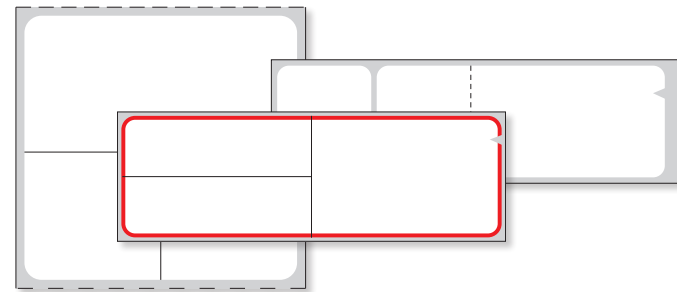

### Accurately Match the Right Patient to the Right Blood with Securline® Bar Code Blood Bands

Positive patient identification is amplified through the use of bar coding which helps eliminate human error and misidentification. These wristbands meet the Joint Commission's National Patient Safety Goal for Blood Transfusions (NPSG.01.03.01) calls for a one-person verification process.



## Timestrip® Blood Temp 10

Button-Activated Blood Temperature Indicator

**PRESS ON FOR ASSURANCE**

Contact Us Today at 888.479.2661

or visit: [www.pdchealthcare.com/bloodtemp10](http://www.pdchealthcare.com/bloodtemp10)

UNITED STATES  
Tel 800.435.4242  
Tel +1 661.257.0233  
info@pdchealthcare.com

27770 N. Entertainment Dr.  
Suite 200  
Valencia, CA 91355

148 E. Stiegel Street  
Manheim, PA 17545

4480 Eastport Parkway  
Port Orange, FL 32137

CANADA  
Tel 800.387.7031  
710 Cochrane Drive  
Markham, ON, L3R 5N7

EUROPE, MIDDLE EAST, AFRICA  
Tel +32 67 89 56 56  
Rue de l'Industrie 17 1400 Nivelles, Belgium

ASIA PACIFIC  
Tel +1 661.257.0233

LATIN AMERICA  
Tel +1 661.257.0233

Distributed by PDC



Where positive identification  
meets safe, efficient patient care.

HC-657-1118



Where positive identification  
meets safe, efficient patient care.



# Timestrip® Blood Temp 10

## PRESS ON FOR ASSURANCE

### Enhance Your Existing Quality Control Process




The Thirty Minute Rule  
≠ temperature monitoring



Timestrip® Blood Temp 10  
monitors real-time  
temperature of the blood



Press ON for the  
easiest indicator

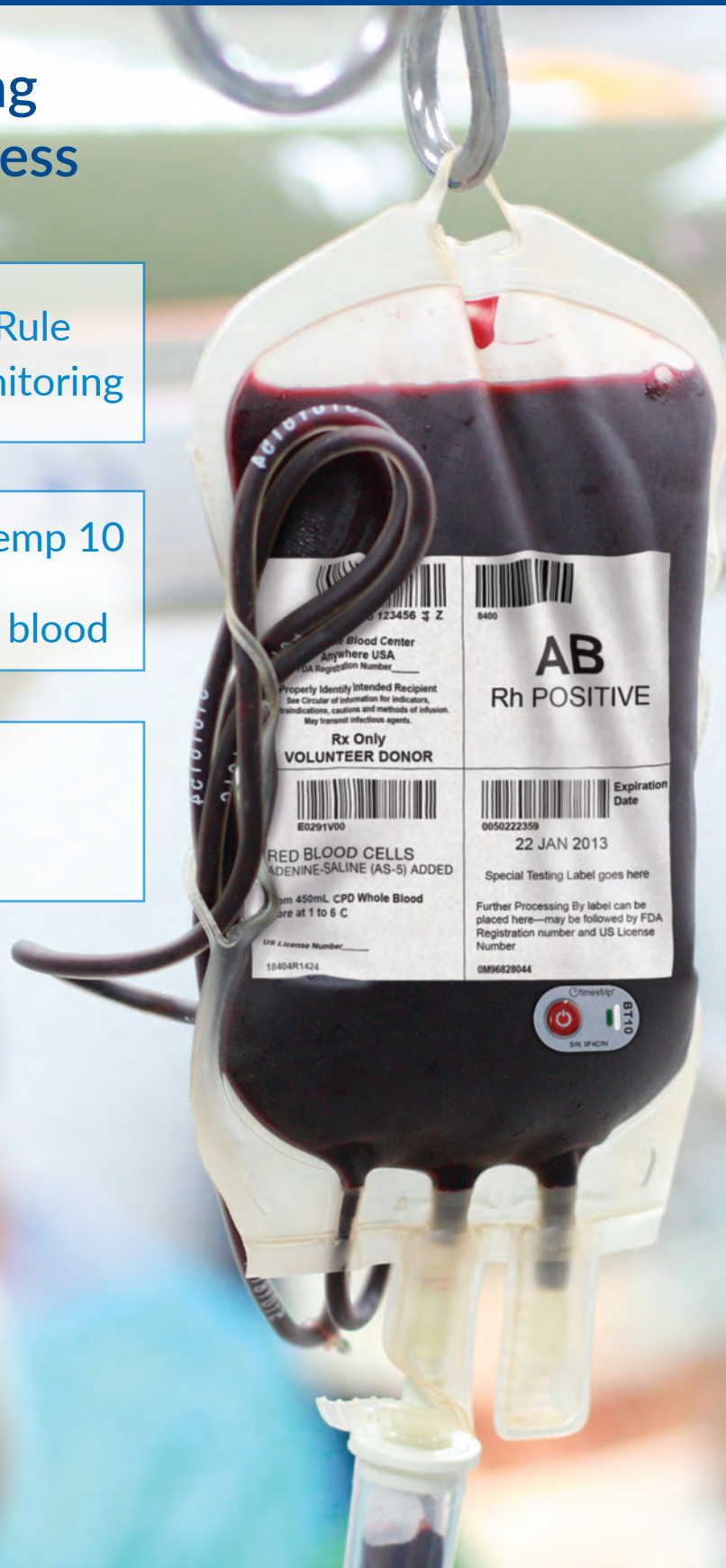

## Simple Quality Assurance for Blood Temperature During Transportation and Storage

### Easiest to Use

- Simple push-button activation
- Indicator requires no special conditioning prior to use
- Cost-effective monitoring

### Quality Assurance

- Maintain quality control in blood transportation and storage
- Real-time data to make quick, safe decisions
- Peace of mind

### Ensure Compliance

- Irreversible proof of meeting AABB Standards and the FDA's CFR 600.15(a) for monitoring temperature of whole blood during transportation and storage
- FDA 510(k) clearance


### Application Is Easy as 1, 2, 3

#### 1 Push the Button



Firmly squeeze the activation button until you feel a pop, then release.

#### 2 Verify Activation

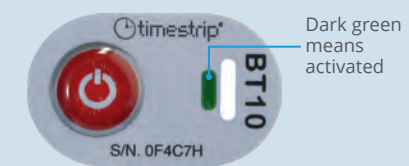

Arming window will change from yellow to dark green immediately. If not, go back to Step 1.


#### 3 Apply to Blood Bag




Remove adhesive liner and adhere activated indicator to the lower third of blood bag.

### At-a-Glance Temperature Monitoring

Upon Activation, Instantly Monitors Blood Temperature

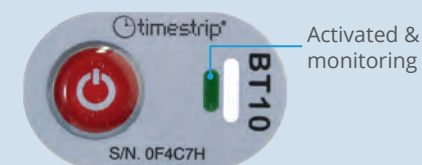

Clearly Indicates When Temperature Breaches 10°C

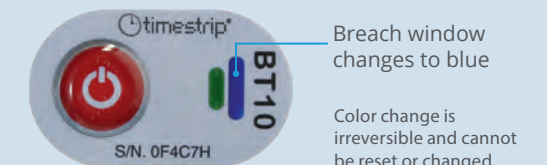

Product Number	Description	Dimensions	Temp Accuracy	UOM
BLOODTEMP10	Timestrip® 10 Blood Bag Temperature Indicator	1-1/4" x 3/4"	±0.5° C, ±1.0° F	100 per box
BLOODTEMP6	Timestrip® 6 Blood Bag Temperature Indicator	1-1/4" x 3/4"	±0.5° C, ±1.0° F	100 per box

Contact Us Today for Free Samples!  
888.479.2661 or  
[www.pdchealthcare.com/bloodtemp10](http://www.pdchealthcare.com/bloodtemp10)

Also Available:  
Timestrip®  
Blood Temp 6  
for Blood Storage!

