

DuraSoft® Laser


Printable Patient ID System Instructions



1 Print patient ID information and bar codes onto the DuraSoft sheet and remove it from your laser printer.



2 As indicated on the DuraSoft sheet, peel the wristband by starting at the adhesive tab.

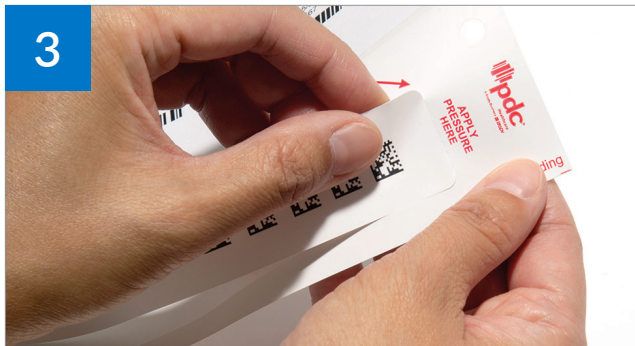

3 As indicated, hold and tear at the perforation to remove the wristband from the sheet.



4 Apply wristband to patient's wrist and secure with adhesive closure. Be sure to leave enough room to ensure comfort and a secure fit.

Contact us today for more information
800.435.4242 | pdchealthcare.com